

Technical Data Sheet

KEY FEATURES

- 95% particle filtration efficiency
- 99.9% bacterial and viral efficiency
- Polyester straps and adjustable nose clip
- Straps – Polyester Elastic Cord
- Meltblown Nonwoven Polypropylene Filter
- Size - Standard

MATERIAL COMPOSITION

- Outer and inner layers – Nonwoven Polyester
- Filter – Meltblown Nonwoven Polypropylene
- Straps – Polyester Elastic Cord
- Nose Clips - Aluminum
- This mask contains no components made from natural rubber latex
- Approximate weight of product: 14 grams

COUNTRY OF ORIGIN

- Made in China

INTENDED USAGE

- High protection mask is intended to be worn by general public or healthcare personnel and covers the user's nose and mouth.
- The mask is used for harsh environments, such as construction sites, cleaning, grinding and dry chemical handling, or other areas with solid particulates and liquid mists.

DO NOT USE FOR

- High protective mask is not for gases, vapors and oil aerosols.
- This mask is not for particulate concentrations exceeding 10X PEL/OEL

TIME USE LIMITATION

- Replace the mask when it becomes dirty, damaged, or difficult to breathe through.



For More Information:

855.615.8633

Orders@J2MedicalSupply.com

© J2 Medical Supply 2021. All rights reserved.

N95 Respirator Masks, AOK Tooling Limited, 3D Mask

Technical Data Sheet - (Continued)

APPROVALS AND STANDARDS

- NIOSH Approved N95 Particulate Respirator
- Meets NIOSH 42 CFR 84 N95 requirements for a minimum 95% filtration efficiency
- NIOSH Approval Number: TC-84A-8449
- FDA: EUA Approval List

SHELF LIFE AND STORAGE

- 5 years from the date of manufacture
- Use by date on box in MM/YYYY format
- Store masks in the original packaging, away from contaminated areas, dust, sunlight, extreme temperatures, excessive moisture, and damaging chemicals
- Store in temperatures between -4°F (-20°C) and +86°F (+30°C) and not exceeding 80% humidity



WARNINGS

- The high protection masks are not intended for reuse and shall be discarded after each procedure. Check your mask's production date. Masks expire two years after the production date, after which they may not provide full protection. Always follow the instructions for use to ensure mask is properly sealed. Not recommended for children under 3.

ORDERING INFORMATION

SKU #	GTIN	DELIVERY	MASKS/BOX	BOXES/CASE	MASKS/CASE
J2N95RM4B	00850020592418	Quoted Upon Request	10	20	200

For More Information:

855.615.8633

Orders@J2MedicalSupply.com

© J2 Medical Supply 2021. All rights reserved.

J2 | MEDICAL
SUPPLY
J2MedicalSupply.com